GRAVIMED PROTEXX

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Table of contents

Preface	8
Declaration of conformity	8
Service life	8
Disposal	8
Identification plate	8
Reuse	8
Warranty	8
Introduction	9
Scope of delivery	9
Intended purpose	9
Indication contraindication	9
Safety instructions	9
Product description	9
Use	9
Cleaning maintenance	10
Storage	10
Disposal	10
Technical data	10

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused. Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at **www.drivedevilbiss.de**

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):
cover

Introduction

The anti-decubitus soft positioning mattress GRAVIMED Protexx has been specially developed for decubitus prophylaxis and to support therapy up to and including decubitus category 3 and is suitable for patients up to 140 kg. The GRAVIMED Protexx is equipped with notches in the base layer and has a soft bed surface to achieve the best possible cushioning and good pressure distribution. The extra soft heel comfort zone offers the best possible relief of the heels. The GRAVIMED Protexx can be used in a standard bed or an electrically adjustable hospital bed / bed frame and is suitable for use in nursing homes and care homes. We expressly ask you to read through the instructions for use carefully so that you obtain the desired medical effect with our product.

Important: The use of an anti-decubitus soft positioning mattress does not replace the regular repositioning of the patient!

Scope of delivery

Please check the contents for damage and completeness. If one of the points listed below is missing, please contact your specialist dealer.

1 x	GRAVIMED foam mattress
1 x	PU cover
1 x	Instructions for use

The GRAVIMED foam mattress was compressed for shipment. Upon receipt, we recommend taking the GRAVIMED foam mattress out of the shipping box and laying it out flat. This will ensure that the foam core can fully unfold and decompress.

Intended purpose

The GRAVIMED Protexx pressure area care mattress has been specially designed to prevent pressure sores for those with a low to high risk and for therapeutic support for categories 1-4 for individuals who can regularly change their lying position themselves or with help and is suitable for patients up to 140 kg. The product works on the pressure-relief principle, which enables the patient to sink into the material and therefore increases the contact area of the body

Indication | contraindication

INDICATION:

- For persons who must lie down for long periods of time due to illness or disability.
- As follow-up treatment for a decubitus which has already healed.
- Decubitus prophylaxis and support of decubitus therapy up to and including category 3.

CONTRAINDICATION:

- In the event of an existing foam allergy.

Safety instructions

- The user (caregiver) must have expert knowledge to be able to estimate the risk of decubitus (e.g. using a recognized scale), assess and evaluate decubital ulcers (decubitus grade according to EPUAP) and be able to perform position changes professionally with the patient in a lying position.
- The maximum permissible patient weight (see technical data) should not be exceeded. The weight information is a recommendation and always refers to the "lying position" of the user.
- Protect the Drive Medical soft positioning mattress from pointed or sharp-edged objects and surfaces. There is a risk of damage!
- Before each use of the Drive Medical soft positioning mattresses, check that the mattresses are in proper and defect-free condition.
- When using side rails on adjustable beds for disabled persons, a sufficient distance between the top edge of the positioning system and the upper end of the bed rail must be ensured (220 mm according to DIN 1970:2000). If necessary, corresponding side rail risers should/must be attached to protect the patient.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Product description

COVER

Flame--retardant according to BS 7175 Crib 5, antifungal, antibacterial, waterproof, breathable, can be disinfected by spraying or wiping, certified according to ÖkoTex standard 100.

GRAVIMED PROTEXX MATTRESS CORE

2-layer antidecubitus mattress with foam layers specially coordinated to each other.

The mattress consists of a 4-cm bed surface made of HR cold foam and a 10-cm base layer made of PU standard foam with special zone notches, certified according to Öko-Tex Standard 100 , product class I; CFC - free; 100% single-origin; covered with a Tricover undercover/mesh cover for easy handling when changing the cover as well as protection from dirt.

Use

Please take the GRAVIMED Protexx out of the box.

- Remove the film using scissors and ensure that you do not damage the GRAVIMED Protexx.

- Check the bed for potential hazards which can damage the soft positioning mattress; see safety instructions.
- Lay the mattress core with the cold foam bed surface facing up in the existing bed.
- The GRAVIMED Protexx has a heel comfort zone which you can identify from the greater number of deeper notches.
- The GRAVIMED Protexx comes covered with an original PU cover from Drive Medical and is therefore ready to use.
- Use only a bed sheet between the cover and user, if possible, and ensure that the bed sheet is also placed without wrinkling.

Changing the cover:

Cover the mattress only with an original PU cover from Drive Medical; ensure that the label with two feet is at the foot of the mattress and is legible from above.

IMPORTANT!!!

Do not use any sheepskins or the like; this can impair the mode of action of an anti-decubitus mattress.

Cleaning | maintenance

GENERAL

In the case of reuse/patient change, the mattress must be cleaned and disinfected according to a validated method recommended by the Robert Koch Institute.

- During processing of the anti-decubitus positioning systems, take note of the medical device law, the „Guidelines for the Processing of Medical Devices“ as well as the „Guidelines for Hospital Hygiene and Infection Prevention“ of the Robert Koch Institute.
- Use only disinfectants and method according to the list of the RKI (Robert Koch Institute) or VAH (Association for Applied Hygiene).
- GRAVIMED soft positioning mattresses are maintenance-free!

COVER

- The cover can be cleaned by hand. For this purpose, use a moist (not wet) cloth and a mild cleaning and disinfectant solution.
- You can wash the cover in the washing machine at max. 95°C using a household detergent. Manual disinfection must be performed according to a validated method. Recommendations of the RKI or VAH should be followed.
- For machine cleaning and disinfection, a validated, automatic method according to RKI or VAH is recommended.
- The mattress cover can be dried in the dryer, however this is not recommended.

Care instructions: Do not use any hypochlorite cleaners!

Please speak to your specialist dealer if you have any further questions about disinfectants and disinfectant options.

GRAVIMED PROTEXX MATTRESS CORE

- Autoclavable according to a validated method according to RKI up to 75°C
- Can be cleaned by hand; for this purpose use a moist (not wet) cloth with warm water and a mild cleaning solution
- **DO NOT SCRUB OR BRUSH**

Storage

- Store the GRAVIMED soft positioning mattresses in a clean and dry condition.
- Store the mattress flat and decompressed and wrapped in film.
- Protect the mattress core from direct sunlight.

Disposal

Please contact your local waste management company regarding professional disposal. Please be aware that the mattress and cover may be carrying microbes. This can lead to transmission of infection.

Please dispose of the overlay and mattress cover such that there is no risk to you and third parties.

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Drive Medical GmbH & Co. KG